


Tips & Tricks: Internal Binding

The most common reason for binding is under-prep, insufficient occlusal clearance, or leaving sharp corners on the prep surface. When binding is present, the simplest correction is to go to "Settings"> "Parameters"> and increase "Adhesive gap" or "Spacer" settings.

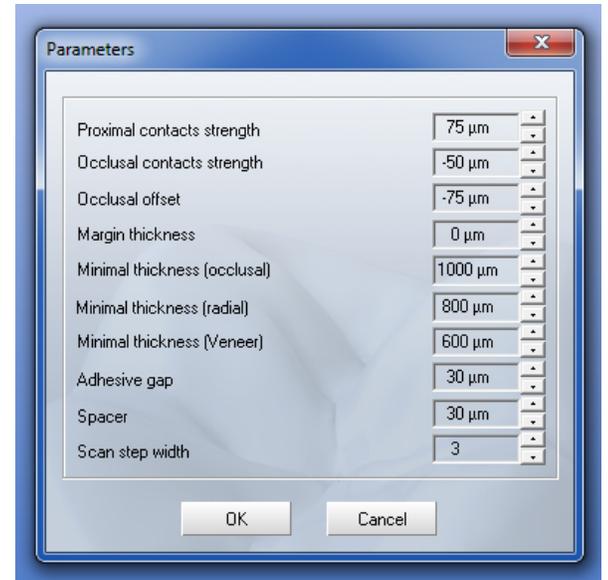

To check "Binding":

- 1) Click the dark blue "Cervical" arrow
- 2) Once you are looking from the underside, simply click "B" while holding on to "Ctrl" key
- 3) Look to see if any porcelain is showing through on the under surfaces
- 4) If you see porcelain showing through, increase either the "adhesive gap" or "Spacer" or both to relief the binding

